



ALL CERAMIC OPTIONS


COSMETIC CHOICE	BRIEF DESCRIPTION	ANTERIOR/POSTERIOR	SINGLE CROWNS	BRIDGES	INLAY/ONLAY	VENEERS	DE-VITAL DARK PREP	CONVENTIONAL CEMENT	ADHESIVE BOND	PREP REQUIRED
	Lifelike metal-free zirconia restorations milled with CAD-CAM for precise fits.	Yes	Anterior/Posterior	Up to 38mm with 2 abutments, up to 47mm with multiple abutments.	Inlay bridges are indicated. System is capable but not commonly used.	No	Yes	Yes	Yes	360 chamfer rounded angles, 10 tapered prep, 1mm axial/1.5mm occlusal
	Sirona's Cerec inLab® is a CAD/CAM system.	Yes	Yes	Yes, 40mm	Yes	Yes	Yes	Yes	Yes	Shoulder/Chamfer
	Leucite-reinforced glass ceramic that provides a highly esthetic solution for single unit restorations that are adhesively bonded.	Yes	Yes	No	Yes	Yes	Yes	Yes	Yes	Circular shoulder or chamfer
	The dentist can select either zirconia or high strength glass ceramic substructure, completed with a single overlay ceramic.	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Circular shoulder or chamfer
	CAD/CAM Zirconia substructure with porcelain overlay, 8 shades, translucent, 5 year warranty	Yes	Yes	1-6 units, splinted crowns, cantilever bridges	No	No	Yes	Yes	Yes	Chamfer 1mm axial/1.5mm occlusal
	Cementable Zirconia-based all-ceramic restoration.	Yes	Yes	3 unit max	No	No	Yes	Yes	No	Shoulder/Chamfer
	Computer-generated core made of dense sintered aluminous oxide or Zirconia for strength.	Yes	Yes	1-4 units Alumina & up to 14 units Zirconia	Yes	Yes, Alumina	Yes	Yes	Yes	360 chamfer rounded line angles 10 tapered prep*
	Electro-statically, layered ceramic utilizing the high strength, bio-compatible Vita InCeram alumina and zirconia ceramics.	Yes	Yes	4 unit anterior & 3 unit posterior	Onlay	No	Yes	Yes	Yes	Shoulder/Chamfer No sharp angles
Laminate Veneers	Ultra-thin porcelain veneers stackable or pressed.	Yes/No	Yes	No	No	Yes	No	No	Yes	Min. prep and ideal esthetics

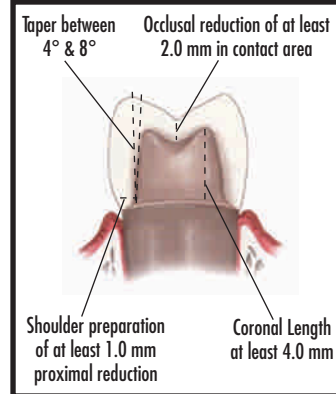

Cementable Ceramic Crown Prep

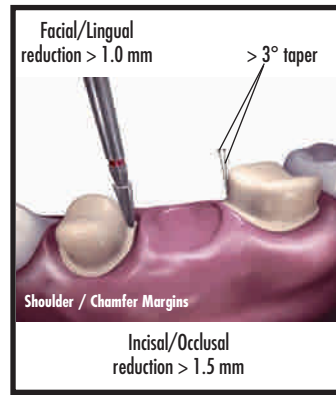

CERCON® Prep Guidelines Crowns/Bridges

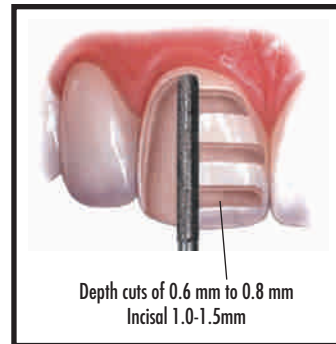

Veneer Prep

PORCELAIN TO METAL

	<ul style="list-style-type: none"> High Noble/Precious - contains at least 40% gold and 60% (total) gold, palladium and platinum. 	Yes	Yes	Case specific	Yes	—	Yes	Yes	Yes	Standard*
	<ul style="list-style-type: none"> Noble/Semi-precious - contains at least (total) 25% gold, palladium and platinum. 	Yes	Yes	Yes	Yes	—	Yes	Yes	Yes	
	<ul style="list-style-type: none"> Predominately Base/Non-precious 	Yes	Yes	Yes	Yes	Yes	—	Yes	Yes	
	Porcelain to "Composite Metal" Gold, platinum & palladium capillary attraction Max. 15 mm span.	Yes	Yes	Yes	Yes	No	Yes	Yes	Yes	Standard*

Implant Services...Call for Information

Call for Lunch & Learn program regarding any services.

* Ask for detailed Manufacturer's prep instruction guide.